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240411:5

# Instruction for use – Catell Multising

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

The brace supports the users arm and shoulder after injury or surgery.

## Application

1. Cut desired length of the strap. Take out 2 pcs of "crocodile hook closures".
2. By using one "crocodile hook", make a loop of one end of the strap and thread the loop around the forearm and place it near the elbow. Picture 1
3. Slide the strap behind the back over the opposite shoulder and towards the hand/wrist. Picture 2
4. At the other end of the strap, make a new loop with the other "crocodile hook" and then place the loop around the wrist. Adjust the correct length for optimal support. Picture 3
5. Make sure that all straps are properly secured.

When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

## Material

Nylon, polyester. Free from latex.

## Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

## Size

By meter.

The product is bilateral.

Single patient use.

## Recycling

Separate plastic and textile. Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB  
Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

240411:5