Bruksanvisning – Catell Armsling Abduction 30°

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240411:5

Instruction for use – Catell Armsling Abduction 30°

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace supports and abducts the users arm and shoulder after injury or surgery.

Application

1. Attach the pillow around the waist using the hook and loop strap and click buckle. Picture 1
2. Open the arm pouches hook and loop closures and place the forearm in the pouch. Picture 2
3. Place the hook strap with buckles in wrist level and the wide short hook strap over the opening of the pouch, near the bend of the arm. Picture 2 Close straps.
4. Pull the fixed pouch-strap behind your back, over the opposite shoulder. Turn through the buckle, attach the hook on the same strap. Thoroughly adjust the correct strap length for optimal support. Picture 3
5. In need of additional hand support, place the short, narrow strap in thumb webbing between thumb and index finger and attach to the pouch. Picture 3
6. Make sure that all straps are properly secured.

When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

Material

Nylon, polyester, polyether foam. Free from latex.

Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

Size	*Dimensions	Size	*Dimensions	Size	*Dimensions
S	28 - 33 cm	M	33 - 38 cm	L	38 - 43 cm

*Measure length of forearm. Measurements should be seen as guidelines.

The product is bilateral.

Single patient use.

Recycling

Separate plastic and textile. Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

240411:5