Bruksanvisning – Catell Armsling Light

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240411:5

Instruction for use – Catell Armsling Light

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace supports the users arm and shoulder after injury or surgery.

Application

1. Make a loop of the strap end and apply it around the forearm near the elbow. Picture 1
2. Pull the strap behind your back, over the opposite shoulder and towards the hand/wrist. Make sure the strap padding is located against the shoulder.
3. Turn the strap around your wrist and adjust the position of the hand by attaching the hook closure on the strap. Thoroughly adjust the correct strap length for optimal support. Wished size of the opening of the loop can be adjusted with the plastic ring. Picture 2
4. Make sure that all straps are properly secured.

When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

Material

Polyamide, polyester. Free from latex.

Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

One size. The product is bilateral.
Single patient use

Recycling

Separate plastic and textile. Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB
Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

240411:5