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Instruction for use – Catell Armsling Smart

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace support and stabilizes the users arm and shoulder after injury or surgery.

Application

The range of the body strap and of the upper arm cuff can easily be adjusted with scissors. Follow the longitudinal seams and cut off any remaining material in connection with any of these.

1. Open the body strap and place it just below the chest. It should have about 20 cm overlap. Close the strap. Remember to turn the strap so that it easily can be opened with the healthy arm / hand. Picture 1
2. Place the upper arm cuff on the upper arm (injured side). Turn the opening so that it easily can be opened with the healthy arm / hand. Close the upper arm cuff and attach it to the body strap. Picture 2
3. Place the wrist cuff on the body strap, at level of the wrist. Make sure the plastic buckle is facing upwards. Then close the cuff.
4. Place the hook of the shoulder strap on center of the back side of the body strap. Place the shoulder strap obliquely over your back and up towards the shoulder on the healthy side.
5. Attach the shoulder strap to the wrist cuff using click buckle. Adjust the length of the shoulder strap. The length of the shoulder strap can easily be adjusted with scissors. Adjust the shoulder pad so that it lies on the shoulder. Picture 3
6. Position the shoulder, arm and hand with care.
7. In need of extra support for the hand, place the short narrow strap in thumb webbing, between thumb and index finger, and attach to wrist cuff. Picture 3
8. Make sure that all straps are properly secured.

When necessary adjust the hook and loop straps. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

Materials

Nylon, polyester. Free from latex.

Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

One size. The product is bilateral.
Single patient use.

Recycling

Separate plastic and textile. Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örnsköldsvik 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

240411:5