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230416:3

# Instruction for use – Combined thumb- and wrist braces

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

The brace supports the users thumb, hand, wrist and forearm after injury, surgery, chronic pain, osteoarthritis or overuse problems. The brace can also be used for preventive purposes in heavy or repetitive movements.

## Application

1. If the brace has a straight splint, take it out of the splint pocket and bend it to the appropriate shape and angle of the users hand and needs. Place it back in the splint pocket. Splints around the thumb can also be adjusted. Splint dent must be turned in towards the palm. Picture 1
2. Place your hand in the brace as illustrated in picture 2 or 4.
3. Close all hook and loop straps so that the brace fits comfortably. Picture 3

Cut off excess hook and loop straps for a cleaner and neater appearance. Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affect blood circulation.

## Material

**Strong:** nylon, elastane. **Stabil:** cotton, polyester, elastane. Free from latex.

## Washing instructions

Close the hook and loop straps and remove the splint, if the brace has one, from the splint pocket. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

## Size

Size	*Dimensions	Size	*Dimensions	Size	*Dimensions
XXS	10-12 cm	M	16-18 cm	XL	20-22 cm
XS	12-14 cm	L	18-20 cm	XXL	22-24 cm
S	14-16 cm				

\*Measure wrist circumference. Measurements should be seen as guidelines.

Single patient use.

**Recycling:** If the product has a removable splint, take it out and sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

230416:3