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Instruction for use – Elbowstrap


Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace supports and relieves the users elbow when the cause is nerve entrapment, after injury, surgery, chronic pain, osteoarthritis or overuse problems. The brace can also be used for preventive purposes in heavy or repetitive movements.

Application

1. Open the hook and loop straps, place the elbow on the brace. Picture 1
 2. Pull the strap so that it encloses the elbow. Close the hook and loop straps. Picture 2
 3. In need of a more stable support, place the rail pocket along the arm. Picture 3
 4. Wrap the remaining part of the band around the elbow and close the hook and loop straps. Picture 4
- When the desired scope of the brace has been achieved it can be pulled on and off without being opened.

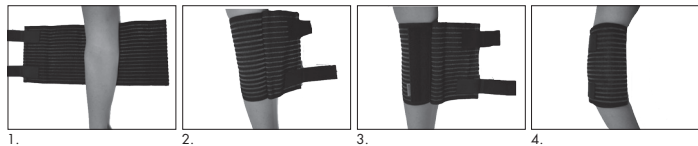Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably, without pinching, chafing or affecting blood circulation.

Material

Polyester, elastane, nylon. Free from latex.

Washing instructions

Close the hook and loop straps and remove the rail pocket. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

Model	Width	Length
Short	19 cm	50 cm
Long	25 cm	60 cm

The product is bilateral. Single patient use.

Recycling

If the product has a removable splint, take it out and sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se