# Bruksanvisning – Epikondylitspänne

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230416:3

# Instruction for use – EPI-strap

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

Lateral or Medial epicondylitis.

The brace supports overused muscles and tendons in the elbow area and will trough this cause painrelief. The brace can be used in combination with, or altered with, a wrist brace.

## Application

### Lateral epicondylitis ("tennis elbow"):

Place the brace with the pad placed towards the outside of the forearm, just below the elbow.

### Medial epicondylitis ("golfer's elbow"):

Place the brace with the pad placed towards the inside of the forearm, just below the elbow.

Open hook and loop, place the brace with the pad as instructed above. In some models, the pad is adjustable. Tighten the strap so that the desired pressure and pain relief are created and the brace fits comfortably. Remember not to tighten too much.

When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affect blood circulation.

## Material

Polyester, polyamide, polyethylene. Free from latex.

## Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

## Size

One size. The product is bilateral. Single patient use.

## Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

230416:3