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Instruction for use - Fingersling

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace supports the users fingers and hand after injury, surgery, chronic pain, osteoarthritis, chronic disease or overuse problems. The brace can also be used for supportive purposes in repetitive movements. The brace can be combined with a wrist brace.

Application

1. Open all hook and loop straps and place the hand in the brace with your thumb in the hole and index finger in closed loop. Place long-, ring-, and little finger between finger straps. Picture 1
2. Close hook and loop strap on backside of the hand.
3. Pull the finger strap in direction towards thumb, place between thumb and index finger, and close hook and loop strap on inside of the hand. Picture 2a, 2b
4. Make sure that the hook and loop straps overlap each other and the brace fits comfortably.

Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affect blood circulation.

Material

Soft: polyester, viscose, elastane, nylon. Free from latex.

Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

Size	*Dimensions	Size	*Dimensions	Size	*Dimensions
S	16-19 cm	M	19-22 cm	L	22-24 cm

*Measure MCP-joints circumference. Measurements should be seen as guidelines. Single patient use.

Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se