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Instruction for use – Fingersplint

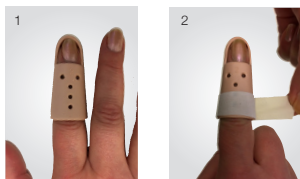
Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The splint is used in purpose to stabilize the distal joint of the finger (DIP-joint).

Application

1. Put the splint on the finger. Picture 1
2. Attach the splint to your finger using skin-friendly tape. Avoid locking the middle joint of the finger (PIP joint). Picture 2

It is important that the splint fits comfortably without pinching, chafing or affecting blood circulation.

Material

ABS-plastic. Free from latex.

Washing instructions

Wash in lukewarm water and gentle cleanser.

Size

Size	*Dimensions	Size	*Dimensions	Size	* Dimensions
1	4 - 4,5 cm	4	5,5 - 6 cm	5.5	6,5 - 7 cm
2	4,5 - 5 cm	5	6 - 6,5 cm	6	7 - 8 cm
3	5 - 5,5 cm				

*Measure distal joint (DIP-joint) circumference. Measurements should be seen as guidelines. Single patient use.

Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

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Manufacturer Catell AB