Bruksanvisning – Handbaserade ortoser

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Instruction for use – Hand based braces

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace supports the user's hand and fingers after injury, surgery, chronic pain, spasticity and/or illness. The brace can also be used for preventive purpose..

Application

1. If the orthosis has a closed band, thread the band over the back of the hand and wrap the hand around the orthosis according to picture 1.
2. If the product has Velcro straps open these and place your hand in the orthosis according to picture 2.
3. Close all Velcro straps so that their hard and soft part cover each other and the orthosis fits comfortably.
4. When using a cone-shaped product, the cone-shape can be used in order to increase the opening grip in the thumb.

Cut off excess hook and loop straps for a cleaner and neater appearance. Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affect blood circulation.

Material

Palm Grip/Finger Contracture: terry, cotton. **Palm Cone:** terry, plastic cone.
All materials are latex-free.

Washing instructions

Close the hook and loop straps and remove the splint, if the brace has one, from the splint pocket. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

Model	Art no	Size	Dimensions of the pillow/cone
Palm Grip	6507	Small	11 x 6 cm
Palm Grip	6508	Large	13 x 7 cm
Palm Cone	5608	One size	13 cm
Finger Contracture	6560	One size	15 x 9 cm

Single patient use.

Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.