Bruksanvisning – Mafra Tumlinda

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Instruction for use – Mafra Tumlinda

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

Simple thumb wrap in soft material, which is well suited for problems in the base of the thumb (CMC-1) such as osteoarthritis, arthritis, pain or overused problems.

Pictures showing example for how to use.

Material

Nylon, rubber. The product contains latex.

Colour

Black, beige

Size

The wrap is 4 cm wide and is available in length of 40 and 50 cm.
Single patient use.

Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch.
Let the brace drip dry.

Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se