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Instruction for use – Smart Strap


Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace supports the users hand and wrist after injury, surgery, chronic pain, osteoarthritis or over-use problems. The brace can also be used for preventive purposes in heavy or repetitive movements.

Application

1. Open the two hook and loop straps. Place your hand in the brace. Picture 1. Note that the picture shows right hand. For application on the left hand, rotate the brace 180° and follow the instructions below.
2. Close all hook and loop straps so that they overlap each other and the brace fits comfortably. Picture 2, 3 and 4.

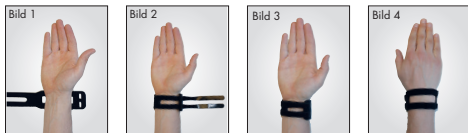When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

Material

Neoprene: synthetic rubber, polyamide. Free from latex.

Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

One size. The product is bilateral.

Single patient use. If the product contains neoprene, use is recommended for maximum 4 hours at a time.

Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.