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# Instruction for use – Wrist Wrap

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

The brace supports the users hand and wrist after injury, surgery, chronic pain, osteoarthritis or over-use problems. The brace can also be used for preventive purposes in heavy or repetitive movements.

## Application

1. Insert the thumb in the hole. Picture 1
2. Hold the wrist in a neutral position and the thumb in palmar abduction during application. Picture 2
3. Wrap the strap towards back of the hand and around your wrist. Close all hook and loop straps so that the brace fits comfortably. Picture 3

When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

## Material

**Stabil:** cotton, polyester, elastane. Free from latex.

## Washing instructions

Close hook and loop straps. Follow the washing instructions below.  
Use a washing pouch. Let the brace drip dry.

## Size

One size. The product is bilateral. Single patient use.

## Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB  
Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

230416:4