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Instruction for use - Prisma braces

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace supports the users hand, wrist and forearm after injury, surgery, chronic pain, osteoarthritis or overuse problems. The brace can also be used for preventive purposes in heavy or repetitive movements.

Application

1. The brace has a preshaped volar splint. It can be adjusted if necessary.
2. Open all hook and loop straps and then close the thumb strap. Place your hand in the brace as illustrated.
3. Close all hook and loop straps so that the brace fits comfortably. Adjust the thumb grip.
4. With the splints in plastic attached, a possibility to adjust dorsal stability is possible by changing existing dorsal metal splints to these.
5. The ulnar support can be strengthening by applying a splint in the open pocket.

Cut off excess hook and loop straps for a cleaner and neater appearance. Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

Material

Stabil Plus: acrylic, cotton, nylon, polyester. Free from latex.

Washing instructions

Close the hook and loop straps and remove the splints from the splint pockets. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

Strl	*Dimension	Size	*Dimension	Size	* Dimension
XXS	10 - 12 cm	M	16 - 18 cm	XL	20 - 22 cm
XS	12 - 14 cm	L	18 - 20 cm	XXL	22 - 24 cm
S	14 - 16 cm				

*Measure wrist circumference. Measurements should be seen as guidelines. Single patient use.

Recycling

If the product has a removable splint, take it out and sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se 230416:4